nosiboo^{*}pro[®]
electric nasal aspirator

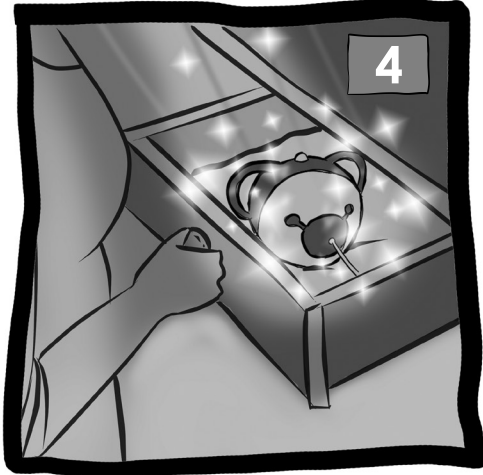
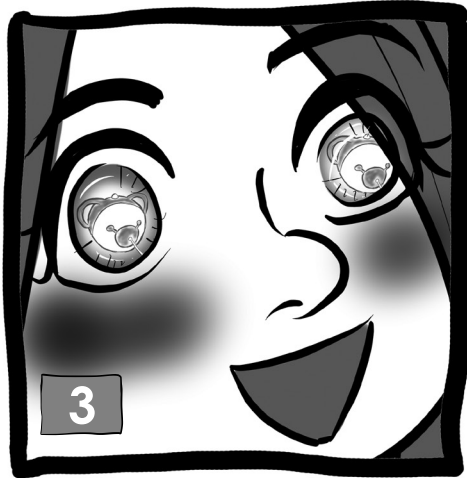
EN

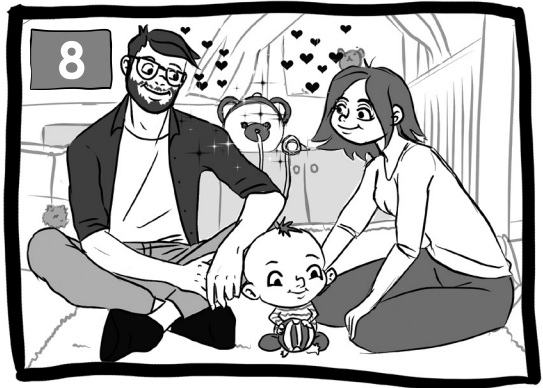
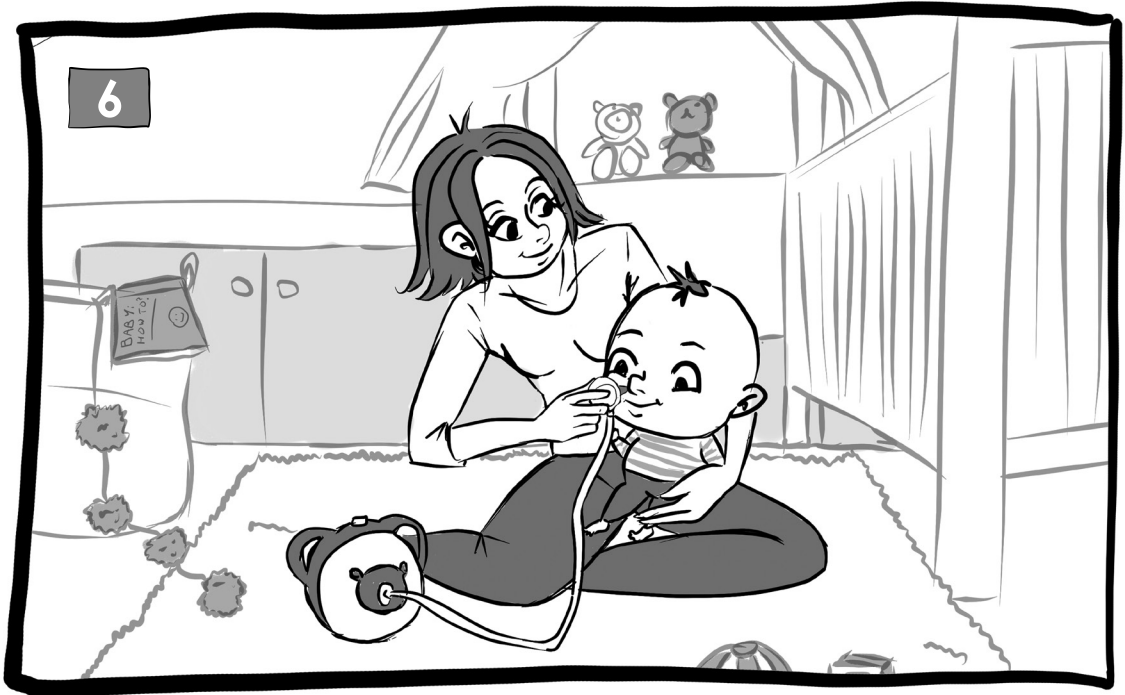
nosiboo^{*}pro[®]
electric nasal aspirator

Before using the Nosiboo Pro, carefully read this guide and the **User Manual** as well! Please keep this guide for future reference.

Supermum & Nosiboo


to be continued...

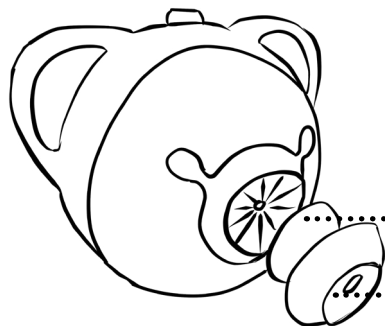IN THE BOX

Quick start guide

User manual

Nosiboo Pro

Connecting tube

Colibri head
+Nose tip

Power cord

Air mesh

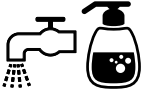
Tube connector

Contact the distributor if any part is missing!

USING THE NOSIBOO PRO




1. Assemble the Colibri head


First clean* and dry the parts of the Colibri head.

*Check the "CLEANING" section below for more details about the proper cleaning method.

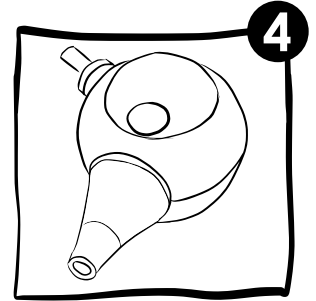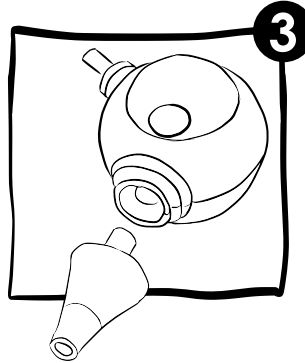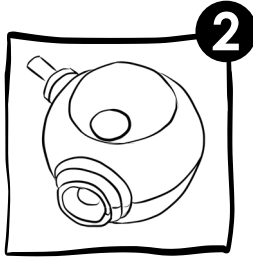
The device can be used for removing the nasal mucus from a child's nose. It cannot be used for any other purpose.

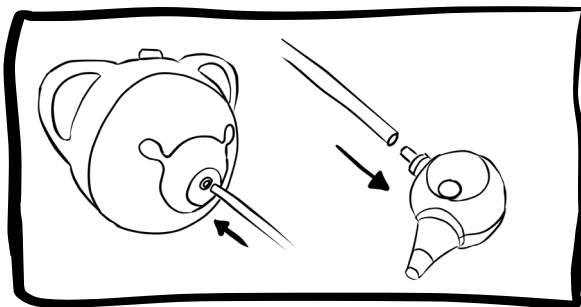2. Assemble the remaining parts

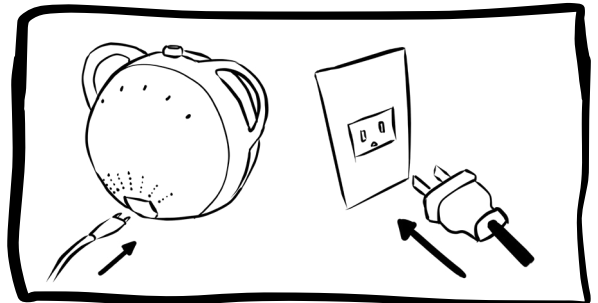a) Attach the connecting tube to the device and to the Colibri head.

b) Attach the power cord to the device and the power outlet.

The device is ready to use.

INSTRUCTIONS FOR USE & SETTINGS

1 Select the correct suction power.

Setting recommendations:

- * min. setting for newborns;
- * middle setting for babies;
- * max. setting for small children and preschoolers.

The maximum suction power was determined by ENTs to guarantee that absolutely no harm can be done by overuse or misuse.

If the mucus is thin, you can use the device at a lower setting. If it is thick you can use the device at a higher setting than normal.

2

Turn on the device by pushing the control button and wait 5 seconds.

3

Hold your child in a stable position.

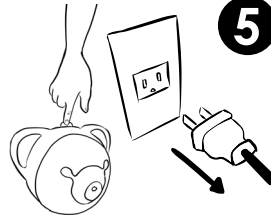
- * Aspirate the nose of infants after laying them into a stable position and hold their head during the process.
- * You may also sit them in an infant bouncer or infant seat.
- * If the child is already able to sit, sit them in your lap. In this case, do not lay the child on their back during the aspiration.

4

Insert the clean, dry nose tip into the child's nostril.

* Move the nose tip around gently to make sure that you get all areas of the nostril.


* In rare instances, it is possible that the nostrils of a newborn are too small for inserting the nose tip. In this case, place the nose tip in a way that the nostril is completely covered. The nasal aspirator's efficiency is enough for cleaning the nasal passages properly.

5

Switch off the device with the control button after use, but don't unplug the device from the electrical outlet until it is completely shut down (min. 5 seconds).

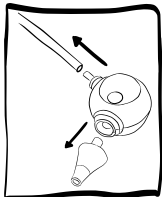CLEANING

* Remove the connecting tube and the nose tip, then take apart the Colibri housing.

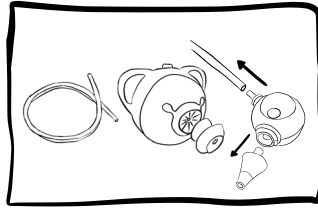
* Wash the parts with liquid soap and lukewarm water.


* You can soak the parts in a disinfection liquid, just make sure the liquid is safe to use for baby equipment.

- * Let the parts dry completely before the next use.

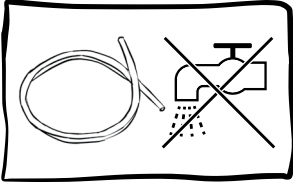
- * It is not allowed to use boiling water, microwave oven, vaporizer or baby bottle sterilizer for cleaning the spare parts because they might deform by heat!


- * Don't wash the connecting tube. Next to proper use, the mucus can't enter the tube so **washing the tube is not necessary**.

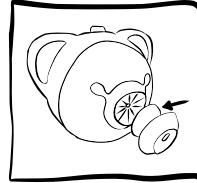- * However, if there is any mucus or liquid in the tube then stop using the device. Clean the tube and let it dry completely. You can wash it under running water and hang it.

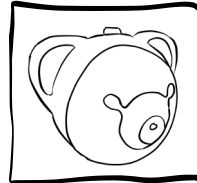
- * The air mesh is there to prevent solid objects and dust from entering the housing.
- * The air mesh is not there to absorb any mucus or liquid.
- * If the air mesh is dirty or clogged, you can replace it.
- * It can function well for a very long time next to the proper usage of the device.


- * Clean the housing with a dry, soft cloth.
- * Do not use any liquids, detergents or chemicals.
- * Keep the device in a space where it is protected from dust, direct sunlight and heat.

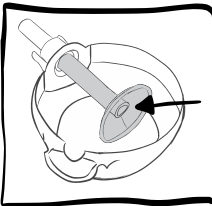
HINTS & TIPS

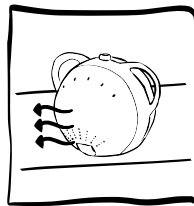- * Do not use the Colibri head and the device without the collared tube.

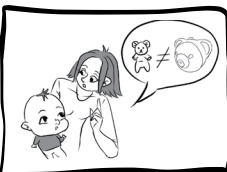- * While in use, the device should be in a stable position, standing on a flat surface.
- * Do not cover the vents on the back of the device while in use. Hot air exits the device through the vents.
- * Make sure the heat exits in a different direction than where your child is.


- * The device should only be used by adults.
- * Do not allow children to play with the device, it is NOT A TOY.

- * Dry, very thick mucus can be dissolved by a mild saline solution. After using it, wait a few minutes, wipe excess discharge with a tissue, then reuse the nasal aspirator as directed.
- * Do not use more than 5 ml of saline solution at once.

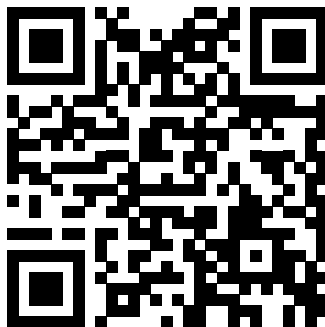User manual

AFTER-SALE SERVICE

For more information about usage, cleaning, and for troubleshooting, see the User Manual.

customers in EU: nosiboo.eu
customerservice@nosiboo.com

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